

ATMOS S 201 Thorax
with hybrid option

Digital cardiothoracic drainage for ERAS[®]- compliant thoracic surgery

*Exactly
what
I need*



ATMOS S 201 Thorax with hybrid option

Perfect for the ERAS® concept*. And more.

Based on clinical evidence, the ERAS® Society in its current guidelines recommends forcing the so-called “rest effect” in the treatment of thoracic patients. In line with the motto “Less is more”, the use of continuous suction should be avoided and the patient’s treatment should be supplemented with conventional forms such as gravity drainage. This is intended to promote the self-closure process in patients with a pulmonary fistula, for example, and significantly accelerate the overall healing process.

However, since most manufacturers of drainage systems offer either digital or conventional solutions, changing the form of therapy also often means changing the device. But not with the ATMOS S 201 Thorax with hybrid option:

This 2-in-1 universal device enables nursing staff to drain with the help of suction and, if necessary, to use the gravity drainage option — in a matter of seconds, without having to change devices by simply switching off the ATMOS S 201 Thorax.



2 in 1 universal device for differentiated drainage therapy



MANY ADVANTAGES FOR YOUR DAILY WORK ROUTINE

- Free choice of therapy form — suction or no suction — without restrictions
- Simple handling with just one click
- Flexible applications in thoracic and cardiac surgery via augmented weaning mode
- Ideal solution for adhering to the current ERAS® Society guidelines in thoracic surgery

LESS EFFORT FOR NURSING STAFF

The ATMOS S 201 Thorax with hybrid option impresses with its integrated “automatic weaning mode” — without any additional effort for nursing staff. During the active drainage process (i.e. the ATMOS S 201 Thorax with hybrid option is switched on), the device monitors the target vacuum set by the user and, thanks to the innovative control system, adapts the activity of the pump to the patient’s needs.

Versatile use via augmented weaning mode



SMALL BUBBLES — BIG IMPACT

With the ATMOS S 201 Thorax with hybrid option, any air leaks become visible during suction-free drainage via bubbles in the water lock of the secretion canister.

	Article number
ATMOS S 201 Thorax	312.1000.0
Hybrid option (ATMOS S 201 Thorax)	312.1090.0

Device switched on = active drainage (“suction”)
Device switched off = gravity drainage (“no suction”)

*** What does the ERAS® concept mean?**

ERAS® is the English abbreviation for “enhanced recovery after surgery”.



info



To learn more about ATMOS Cardiothoracic Drainage, scan the QR code or visit our website at: www.atmosmed.com



ATMOS MedizinTechnik GmbH & Co. KG

Ludwig-Kegel-Str. 16
79853 Lenzkirch / Germany

Phone +49 (0)7653-689-0
Fax +49 (0)7653-689-190
E-mail info@atmosmed.com

www.atmosmed.com



Scan the QR code to go to the ATMOS S 201 Thorax with hybrid option web page.